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---

Instruction Manual  
**Blowpipe B16**

TeleDart GmbH & Co. KG  
Obere Heide 8  
67368 Westheim  
Germany

Phone: +49 (0) 6344 939765  
Fax: +49 (0) 6344 939890  
E-Mail: [info@teledart.com](mailto:info@teledart.com)  
[www.teledart.com](http://www.teledart.com)

## **Rechtliche Hinweise / Legal notes**

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*•TeleDart GmbH & Co. KG accepts no liability for damages caused to persons or property due to improper or negligent handling of blowpipes, darts or needles. Please adhere to all regulations applicable to your country in regard to the handling of blowpipes, darts and needles as well as animal protection and welfare. Please follow strictly all safety advice of the drug producer.*

*Keep the blowpipe, darts and needles away from children and also do not aim on other persons. Only use the blowpipe, if you are sure that no other persons are in the danger zone.*

*Please contact your local authorities for legal information and premises on the tranquillizing or medical treatment of animals in your country.*

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**All information on the handling of TeleDart® dart syringes have to be taken from the according separate manual!**

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## Scope of delivery

### **Single Blowpipe “B16”**

- 1 Blowpipe B16 Ø 16 mm, length 1 m With removable mouthpiece
- 1 Cleaning kit

### **Blowpipe kit “Kit B16”**

- 1 Blowpipe B16 Ø 16 mm, 1 m length with removable mouthpiece
- 1 Carry bag
- 3 Blowpipe darts 3cc
- 2 Blowpipe darts 5cc
- 3 Injection needles 1,5 x 30 mm
- 2 Injection needles 1,5 x 38 mm
- 3 Training darts 3 cc
- 2 Training darts 5 cc
- 1 Filling & care kit
- 1 Blowpipe cleaning set
- 10 Special flight stabilizers BST16

### **Accessories**

(not included in scope of delivery)

### **Blowpipe extension “BP16”**

Ø 16 mm, 1 m length

The blowpipe extension has a quick coupling and enables the shooting at larger distances.

## Schussdistanz / Shooting distances

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Depending on the size of the dart syringe, the surrounding conditions or the shooting performance of the user the following distances can be achieved:

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Following dart syringes and needles can be used with the blowpipe B16:

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BD1	1 ml
BD2	2 ml
BD3	3 ml
BD5	5 ml

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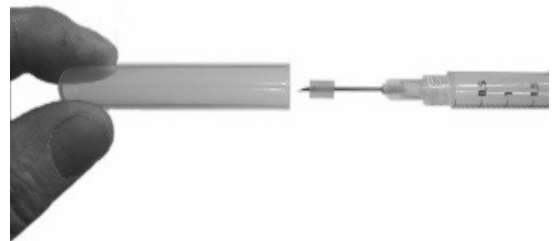
■ Take off the removable part of the safety mouthpiece.

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■ Take off the safety cap from the dart syringe.

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■ Push the completely filled and pressurized dart syringe into the blowpipe.

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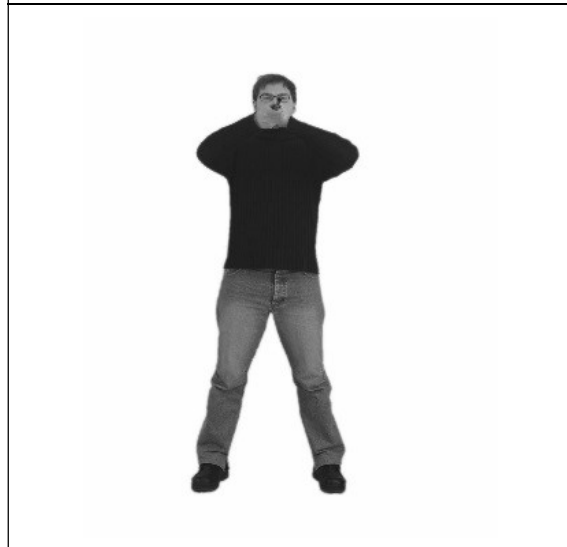
■ When putting the mouthpiece back onto the blowpipe, the dart syringe is placed in its final position.

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
■ Stand in a frontal position to the target animal.

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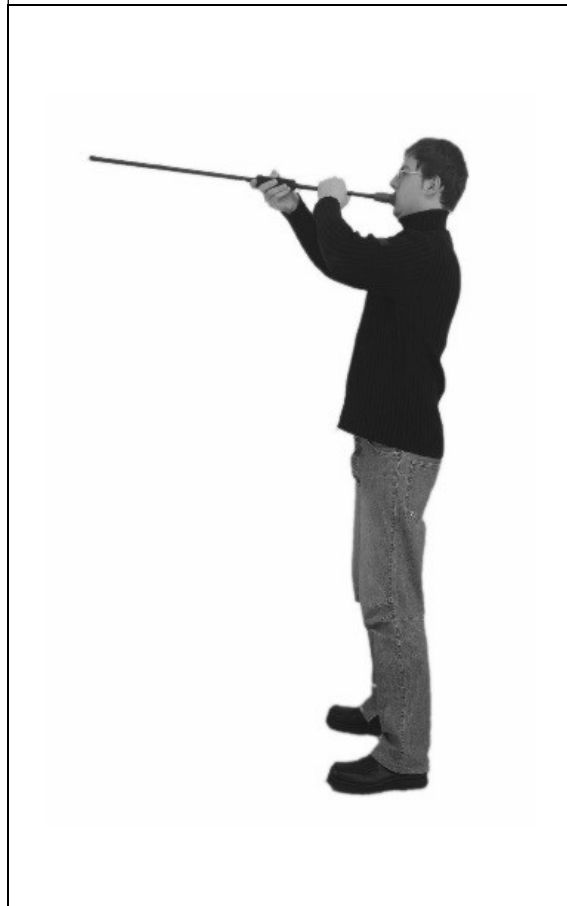
■ Press the mouthpiece close to your mouth to avoid air to escape and take a deep breath. Then exhale the air as strong and as fast as possible. Keep on blowing until the dart has left the blowpipe.

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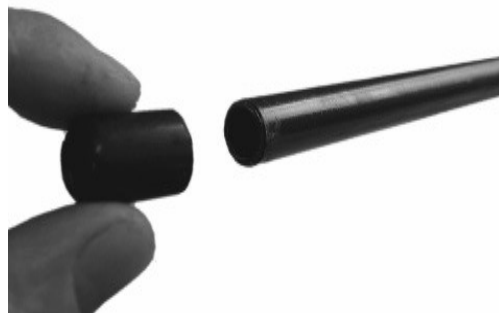■ Remove the cap from the front part of the blowpipe.

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■ Grease the front part of the blowpipe.

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■ Put the blowpipe into the extension and with turning movements push both tubes together as far as it will go.

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Before using your blowpipe in the field for the first time and also later you should take half an hour from time to time to practice with your blowpipe on a target which is comparable to the animal's body. To work out your routine you also should practice regularly. Note: the more often you practice the better and safer your shooting results will be.

For practice simply use the training darts syringe of your choice.

Very soon you will become familiar with your blowpipe.

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Clean the mouthpiece with hot water and soap or dish liquid.

Clean the blowpipe and the extension with the dry cleaning tool by pulling it several times through your blowpipe.

**If using a blowpipe Extension** the top of the blowpipe (15 cm) must be greased to guarantee an optimal coupling with the extension.