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disposable drug syringe
(FS3/FS5)
disposable drug needle (FN)

Filling the TeleDart® blowpipe darts


The following steps will show you how to fill the dart correctly.

We recommend to practise the filling of darts several times until you become familiar with it.

- Take off flight-stabilizer and safety cap.
- The red air plunger must be loose.
- Note: If the air plunger is not loose, push venting pin VP inside the air chamber to loosen the plunger.

- Fill the air filling syringe (with connector CON) with air.
- Put the air filling syringe onto drug chamber (black plunger) of your dart.
- Hold dart upright (air chamber shows to the top) and press black drug plunger to the other end of drug chamber.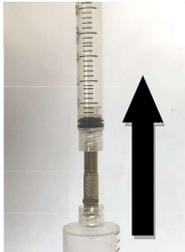

Preparing the needle


- The needle has got a lateral opening on the side. Carefully push sealing sleeve in turning movements onto needle until the lateral opening is completely covered.

The sealing sleeve must not be damaged. Only use it once!

Filling the drug

- Fill the drug into drug chamber (black plunger) of the dart.
- If the dart is underdosed (e.g. only 2ml of solution in a 3ml dart or similar), it is recommended for optimal ballistics to either fill the medication chamber with saline-solution or to move the black plunger with the air filling syringe so that the only desired volume is free.
- Remaining air bubbles in the drug chamber are harmless and do not influence the function.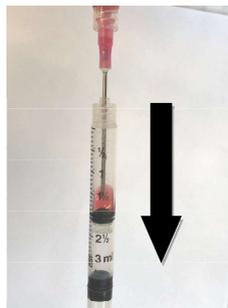

The syringe tipp always must be dry and free of grease!

- **Blowpipe needle:** Screw on the blowpipe needle onto the drug chamber of the dart (Luer-Lock).


- Now cover the needle by putting the safety cap back onto the dart.

The safety cap prevents the loss of drugs if the needle is leaking.

- Fill air filling syringe (with connector CON) with air and connect it with the air chamber (red plunger) of your dart.
- Hold dart exactly upright with needle showing to the top!
- Now press air into the air chamber until you feel a strong backpressure. In that moment keep the air filling syringe pressed and **at the same time** take it off in a fast movement.

The higher the pressure, the better the dart releases.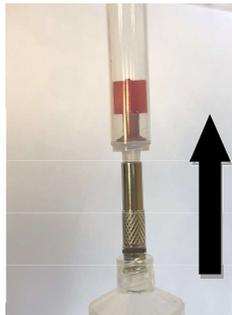

Pressure control:

- The air plunger must not be loose! Test it by flipping the air chamber with your finger. If the membrane releases and becomes loose inside the air chamber, do not use the dart. Please repeat the process and try to press more air into the air chamber this time.

- Put the flight-stabilizer back onto the dart. Use GBST plastic stabilizer for use with blowpipe B16.
- The dart is now ready for use.
- Do not take off the safety cap until you start treatment.

Please handle filled darts always very carefully!

Unloading the TeleDart® blowpipe darts

What to do, if you want to unload a filled blowpipe dart?


- Take off the flight-stabilizer.

- Deflate the dart by pushing the venting pin VP into the air chamber. The red air plunger must be loose.

- If you are sure that the dart is de-aerated, hold it with the needle to the top.
- Take off the safety cap.
- Unscrew the needle.
- Put a disposable drug needle onto the drug chamber of the dart.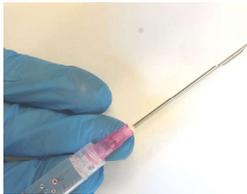

- Put the air filling syringe (with connector CON) onto the drug chamber of the dart.
- Use a drug bottle (preferably with rubberised seal) and
- Press the drug into the bottle.

Please take into consideration, that the concentration of the drug has been changed by adding saline-solution. Therefore, we recommend to store these dosages always separately and label them exactly.

Cleaning of empty TeleDart® blowpipe darts and needles

TeleDart® blowpipe darts and needles can be used a long time, if they are cleaned carefully after use.

Cleaning the needle:

- Take silicone sealing sleeve off the needle.
- Unscrew needle off the blowpipe dart.
- Hold needle under running water, to clean it from blood/debris.
- Fill the air filling syringe (with connector CON) with water.
- Press water through the needle.

Cleaning the blowpipe dart:

- Put on flight-stabilizer.
- Fill air filling syringe (with connector CON) with water.
- Put the air filling syringe onto the drug chamber.
- Press water into the drug chamber several times to rinse all residues.

Note: Never clean the air chamber of the dart. It always has to be kept dry!

Care of TeleDart ® blowpipe darts and needles

To assure the function of the dart, we recommend cleaning the black drug plunger with silicone oil (SO) after every use:

- Put the flight-stabilizer onto the dart.
- Softly press air into the dart by using air filling syringe to move black plunger to the middle of the drug chamber.
- Be careful not to contact the outside of the dart's top with oil.
- Transfer a drop of silicone oil into the drug chamber.
- Again, use the air filling syringe and move the black drug plunger several times inside the drug chamber to disperse the silicone oil.

- Leave the oil several hours in the dart syringe before using it again!

The tip of the dart must always be free of oil. Only use original TeleDart ® silicone oil.

Storage of TeleDart ® blowpipe darts

Always store TeleDart ® blowpipe darts unfilled and deflated. Though the darts consist of a continuous-stable plastic material, even under a dry storage with temperatures from 0- 30°C they should be used no longer than 36 months. Protect your darts against UV-rays as this may influence their lifetime.