

## Pole Syringe JS-12 manual



The pole comes with an already installed syringe with 12cc.



For filling the syringe remove the aluminium cap and check if the syringe moves flawlessly.



Put on a needle on the syringe and lock it by turning. Then fill the syringe as usual with the needed amount of medicine.



Secure the needle by re-installing the aluminium cap you have removed before. Take care not to bend the needle or puncture yourself!



As next step, screw the complete head to the base part of the pole, as shown in the picture below.



Now the pole syringe is ready to use.

# Instructions for telescopic function



**Attention!** The pole must be held in place before the telescopic function may be used. Take care, that no one is around or in the way of the pole.

Hold the pole and push the green button on the pole (see arrow) and pull or push the pole to the according length. For fixing the pole stop pushing the knob and adjust the length until it locks. Rechk for good fixing.

Clean the mechanism from time to times with clear water.

## Take utmost care:

Do not use the pole syringe close to power fences, high voltage power lines or similar. There might be the danger for an electric shock for you or the animal.



## Replacing the syringe:

There are 12 cc syringes provided.

To replace the already mounted syringe take off the needle and syringe main body and discard them. Also discard the black rubber cap of the upper aluminium part.

Use the new syringe, pull the piston out and mode the black rubber over to the aluminium part. Push over the new syringe main body. Discard the left over plastic piston of the new syringe.

Filling as described above.

## Warning! Please read! Use at your own risk.

The use of this device is entirely at the discretion of the user.

The user accepts complete responsibility for the use of this device.

The user must become fully aware of its operation before using. Teledart assumes no responsibility or liability for any damage(s) or accident(s) resulting from the use of its products whatsoever.

- Keep clear of electrical wires.
- Safe use of this tool includes but is not limited to training, practice, knowledge and being familiar with the animals you are attempting to handle.
- Do not put yourself or other Person at risk. All animals can be unpredictable.

## Technical Details:

Minimum Length 138 cm

Maximum Length 246 cm

Weight 1200 gram

Material: glassfiber, aluminum, end with a soft grip.

## teledart GmbH & Co. KG

Obere Heide 8

67368 Westheim / Germany

Tel.: +49 (0)6344 - 939765

Whatsapp: 01575 3254319

E-Mail: [info@teledart.com](mailto:info@teledart.com)

Internet: [www.teledart.com](http://www.teledart.com)