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Instruction manual for syringe darts

TeleDart GmbH & Co. KG
Obere Heide 8
67368 Westheim
Germany
Tel.: +49 (0) 6344 939765
Fax: +49 (0) 6344 939890
WhatsApp: +49 (0) 1575 3254319
E-Mail: info@teledart.com
Internet: www.teledart.com

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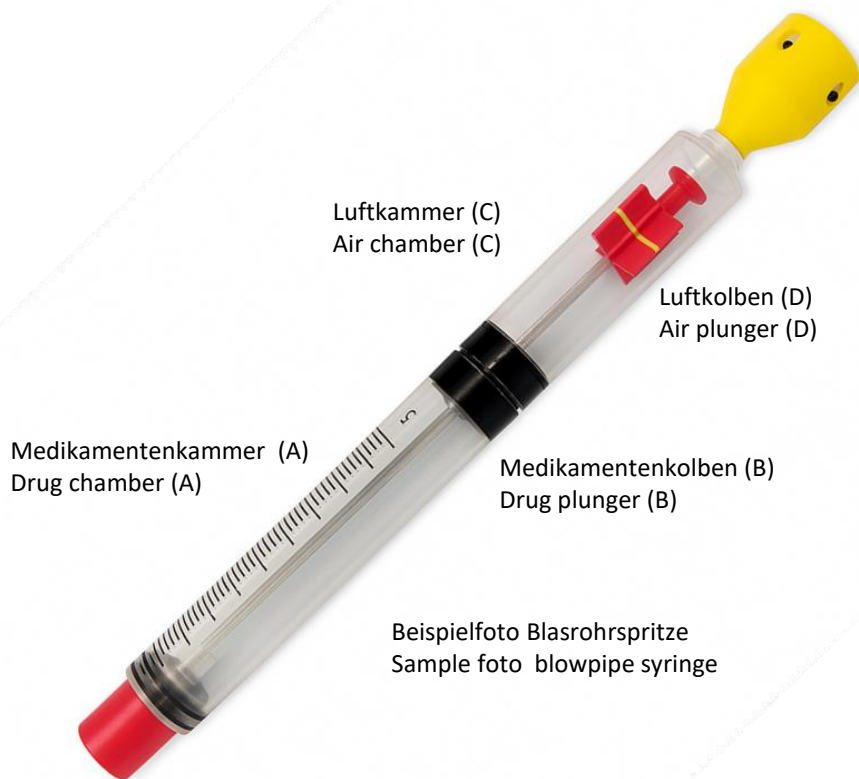
Legal notes: TeleDart GmbH & Co. KG accepts no liability for damages caused by improper handling of darts. Please adhere to all regulations applicable to your country in regard to the handling of weapons/blowpipes/ tranquilisers as well as animal protection and welfare. Please follow strictly all safety advice of the drug manufacturer.

Dispose only empty and clean darts without needle.

Dispose needle cleaned or disinfected in suitable safety box.

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- Remove flight stabilizer (E)
- Check if red air plunger (D) can move freely

! If the red air plunger (D) is stuck:

Pierce with venting pin (G) and hold syringe vertically so that the red plunger (D) drops to the middle

2. Prepare Pressurizing

- Fill pressurizing syringe (F) with air
- Place pressurizing syringe (F) on medication chamber (A)
- Hold syringe dart vertically, (air chamber (C)) pointing upwards
- Use the air to push black plunger (B) into center of syringe

3. Prepare needle

- The needle has got a lateral opening on the side.
- Slide silicone sleeve onto the needle with slight twisting onto the needle until:
 - The lateral opening is centered
 - Completely sealed
- Use the sealing sleeves only once

4. Filling medication

- Fill the drug chamber (A) with medication

Note on dosage:

- In case of underdosing (e. g. 2 cc in a 3 cc syringe):
 - Fill up with NaCl
 - Or adjust plunger
- Lower weight can affect flight characteristics

Small air bubbles are harmless (do not impair function and are not dangerous for the animal).

5. Attach needle

- Screw on needle (Luer-Lock system) or push on needles **without** LuerLock with needle tool ST
- Place protective cap over the needle

The protective cap protects you and prevents medication loss in the event of leakage.

6. Pressurizing

1. Fill pressurizing syringe (F) completely with air. Place it on air chamber (C) from below
2. Hold dart syringe fully upright and place it on the pressurizing syringe (F) from above
3. Press air from the pressurizing syringe (F) into the air chamber (D), by placing the pressurizing syringe (F) on a firm surface, press the dart syringe down until you feel clear pressure and the plunger can't be moved any further.
4. Maintain pressure while holding the air plunger of pressurizing syringe (F) to prevent loss of pressure and quickly remove the dart syringe.

7. Finishing

- Attach the appropriate flight stabilizer (E)

The dart is now ready for use.

After sedating the animal

1. Check & remove the dart syringe

- Check if medication has been completely injected.
- Remove flight stabilizer (E)
- Release any remaining pressure: loosen air plunger (D) with the venting pin (G)
- Remove dart syringe from the animal's body, replace protective cap and stabilizer

Tip: Clean the dart syringe and needle immediately after use, as dried medication debris impair their function.

Unloading the dart syringe

1. Preparation

- Remove flight stabilizer (E)
- Release residual pressure (as above)
- Remove the protective cap
- Prepare the medication chamber (with rubber membrane)
- Pierce a second hole in the membrane (prevent vacuum)
- Place pressurizing syringe (F) on the pressure chamber
- Press back the medication

Note: Concentration may have changed due to saline solution → Store and label returns separately.

Reuse – Check before use

- Check dart syringe and needles for deformations / cracks / durability of the needle

If damaged: do not use, because dart might not fly properly or loose medication!

Cleaning dart syringe & needle

1. Dart syringe

- Unscrew the needle
- Fill pressure syringe (F) with water and attach it to the medication chamber (A)
- Press water into the chamber → allow it to escape again by overpressure
- Repeat several times if necessary
- Release any remaining pressure from the air chamber

Caution: Do NOT clean the air chamber – always keep it dry!

2. Needle

- Remove and dispose of silicone sleeve
- Rinse under running water (blood, tissue)
- Flush with water or saline solution

Check side outlet → repeat if necessary

Care of empty dart syringes

- Maintain the medication plunger (B) after each use with silicone oil
- Push the plunger (B) toward the center using the pressurizing syringe (F)
- Put 1 drop of silicone oil into the drug chamber (A).
- Rotate dart syringe and move the plunger back and forth.
- Leave to take effect for several hours.
- Allow excess oil to drain off (tip pointing downwards).
- Press air out of the chamber until the plunger reaches its end position.

Do not oil the air plunger (D)! The needle seat must not come into contact with oil! Always keep tip of dart syringe free of oil so that the needle does not come loose.

Storage of dart syringe

- store unfilled and without pressure.
- Store in a dry place at 0–30°C for a maximum of 36 months.
- Protect from UV radiation!

Blank lined paper for writing.